

1.5 Chambers and nozzles

The Microfine jet blaster offers optimum surface quality for blasting different materials with abrasives, aluminium oxide, glass beads and combined abrasives from 25 to 250 µm.

m	1. Chamber blue	2. Chamber red	3. Chamber white
Device with 2 chambers	25–125 µm	nonexistent	125–250 µm
Device with 3 chambers	25–75 µm	75–150 µm	150–250 µm

List of nozzles and chambers:

	1. Chamber blue	2. Chamber red	3. Chamber white
Device with 2 chambers	1,2 mm	nonexistent	1,5 mm
Device with 3 chambers	0,8 mm	1,2 mm	1,5 mm

Always use recommended nozzle size for chamber 1. Using other nozzle sizes than indicated in the chart for the other chambers does not provide optimum blasting results.



Fig.: Details on SANDUBLAST 2



Fig.: SANDUBLAST 3